

Instructions for use

AF™ Baby File

[**PRODUCT NAME**] AF™ Baby File

[**SPECIFICATION**] Refer to specification sheet

[**COMPOSITION**]



The rotary file can be divided into two parts:

attachment section, made of copper; and working parts, made of NITI.

[**APPLIED RANGE**]

The instruments are meant to be used in a clinical/hospital environment, by qualified personnel, according to the correct operative procedures (gloves, face mask, rubber dam). And the instruments are indicated for the endodontic retreatment, shaping and cleaning of the root canal system.

- The first file system specially designed for primary teeth.
- AFTM-HWire technology.
- Triangular cross section design.
- Improved resistance to cyclic fatigue.
- Advanced tip processing & avoid ledge formation.

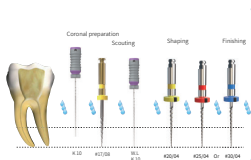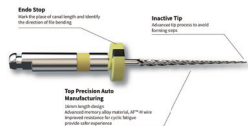

1. Negotiate the coronal 2/3 of the canal with K file 10.
2. Use Open File to prepare the coronal third and getting a straight line access.
3. Irrigate the canal.
4. With the use of apex locator, negotiate the canal with K file 10 in a watch-wind motion to full working length, to get a patent canal pathway.
5. Irrigate the canal.
6. Use file (#20/0.04) in a pecking motion to full working length.
7. Irrigate the canal.
8. Use file (#25/0.04) in a pecking motion to full working length.
9. Irrigate the canal.
10. You can stop at this file, or if the canal needs more preparation, you can proceed with file (#30/0.04) in a pecking motion.
11. Irrigate the canal.

[**CONTRAINDICATION**] Do not use in patients allergic to Nickel.

[**STERILIZATION**]

Products must be sterilized before use.

- 1) Rinse the device with Normal saline (0.9% NaCl) for 10 seconds;
- 2) Dry the device with a dental air gun.
- 3) Steam sterilization at 134°C for 18 minutes;

[**CAUTIONS**]

- 1) The file tip can not beyond the apical when you are shaping the canal.
- 2) It can not work if there is any incorrect operation: use the wrong technique when operating; the angle of rotation beyond 180°; use the files without the sequence; performing the ledges; without patency and excessive force to the canal.
- 3) Frequently clean the spires; irrigate frequently and abundantly; follow the advised settings RPM - Torque Start the procedure after scouting; Work with plenty of irrigant into the root canal.
- 4) Symbols: There is information about the manufacturer, product name, specification, color model, quantity and production date printed on the package.

Instructions for use

AF™ Baby File



[**Storage and Transportation**]

- 1) Keep dry.
- 2) Avoid storing at high temperature, and direct sunlight, humidity shall be less than 80%.
- 3) Avoid from moisture and corrosion gas.
- 4) Do not store with toxic, chemical and hazardous substances.
- 5) Store it at room temperature.

[**PACKAGE**] Individual package

[**PRODUCTION DATE**] Refer to seal or label on the package.

[**Expiration Date**] 5 Years

[**Symbols Instruction**]

	Manufacturer		Date of manufacture
	Authorized Representative in the European Community		Use-by date
	Caution		Batch Number
	Do not reuse		Keep dry
	Sterilizable in a steam sterilizer (autoclave) at the specified temperature		Keep away from sunlight
	Consult instructions for use		EC mark and Identification number of Notified Body
	Manufacturer Logo		

[**Registered Company**] Shanghai Fanta Dental Materials Co., Ltd

[**Registration Address**] Room 1302, building 2, No. 6965, Guangming Puxing Road, Qingcun Town, Fengxian District, Shanghai, China

[**Tel:**] +021-64033133

[**After-sales Service Company**] Shanghai Fanta Dental Materials Co., Ltd

[**Europe Authorised Representative Address**] SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Contact: Yana Zhang (Ms.)

[**Tel:**] +31 (0) 2021 11106

[**Prepared Date for Instruction**] 20200325