

# Instructions for use

## Max Cleaning

[ **PRODUCT NAME** ] Max Cleaning

[ **SPECIFICATION** ] Refer to specification sheet


[ **COMPOSITION** ]

The rotary file can be divided into two parts:  
attachment section, made of copper; and working parts, made of NiTi.

[ **APPLIED RANGE** ]

The instruments are meant to be used in a clinical/hospital environment, by qualified personnel, according to the correct operative procedures (gloves, face mask, rubber dam). And the instruments are indicated for the endodontic retreatment, shaping and cleaning of the root canal system.

- Minimal invasive file with maximum cleaning.
- Minimal stress applied on dentinal walls.
- Super elasticity with extreme flexibility.
- Maximum cleaning of the canal with smear layer removal.

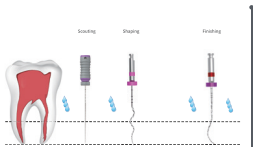

1. Irrigate the canal.
2. With the use of apex locator, negotiate the canal with C File 10 in a watch-wind motion to full working length, to get a patent canal pathway.
3. Irrigate the canal.
4. Use Max 3 in a pecking motion (Pecking motion: in and out motion for a depth of 3 mm) till reaching the working length. The motion of the file is on rotation movement (speed 800RPM, Torque 1N).
5. Irrigate the canal.
6. To ensure better cleaning of the apical third and agitation of irrigation, use Max 1 with the same manner and settings mentioned in step 4.
7. Irrigate the canal.

[ **CONTRAINDICATION** ] Do not use in patients allergic to Nickel.

[ **STERILIZATION** ]

Products must be sterilized before use.

- 1) Rinse the device with Normal saline (0.9% NaCl) for 10 seconds;
- 2) Dry the device with a dental air gun.
- 3) Steam sterilization at 134°C for 18 minutes;

# Instructions for use

## Max Cleaning

[ **CAUTIONS** ]

- 1) The file tip can not beyond the apical when you are shaping the canal.
- 2) It can not work if there is any incorrect operation: use the wrong technique when operating; the angle of rotation beyond 180°; use the files without the sequence; performing the ledges; without patency and excessive force to the canal.
- 3) Frequently clean the spires; irrigate frequently and abundantly; follow the advised settings RPM - Torque Start the procedure after scouting; Work with plenty of irrigant into the root canal.
- 4) Symbols: There is information about the manufacturer, product name, specification, color model, quantity and production date printed on the package.

[ **Storage and Transportation** ]

- 1) Keep dry.
- 2) Avoid storing at high temperature, and direct sunlight, humidity shall be less than 80%.
- 3) Avoid from moisture and corrosion gas.
- 4) Do not store with toxic, chemical and hazardous substances.
- 5) Store it at room temperature.

[ **PACKAGE** ] Individual package

[ **PRODUCTION DATE** ] Refer to seal or label on the package.

[ **Expiration Date** ] 5 Years

[ **Symbols Instruction** ]

|  |                                                                             |  |                                                    |
|--|-----------------------------------------------------------------------------|--|----------------------------------------------------|
|  | Manufacturer                                                                |  | Date of manufacture                                |
|  | Authorized Representative in the European Community                         |  | Use-by date                                        |
|  | Caution                                                                     |  | Batch Number                                       |
|  | Do not reuse                                                                |  | Keep dry                                           |
|  | Sterilizable in a steam sterilizer (autoclave) at the specified temperature |  | Keep away from sunlight                            |
|  | Consult instructions for use                                                |  | EC mark and identification number of Notified Body |
|  | Manufacturer Logo                                                           |  |                                                    |

[ **Registered Company** ] Shanghai Fanta Dental Materials Co., Ltd

[ **Registration Address** ] Room 1302, building 2, No. 6965, Guangming Puxing Road, Qingcun Town, Fengxian District, Shanghai, China

[ **Tel:** ] +021-64033133

[ **After-sales Service Company** ] Shanghai Fanta Dental Materials Co., Ltd

[ **Europe Authorised Representative Address** ] SUNGO Europe B.V.

Olympisch Stadion 24, 1076DE Amsterdam, Netherlands

Contact: Yana Zhang (Ms.)

[ **Tel:** ] +31 (0) 2021 11106

[ **Prepared Date for Instruction** ] 20200325